

Description „End of Session, Conclusion and returning to the starting point“

Every single training, hypoxic test and interrupted sessions **ends finally with a conclusion**. Only complete Manual and Biofeedback Sessions could go directly to conclusion mask. After Tests and interrupted sessions a 3 Minute Relaxation Phase is always offered.

Training Session

Manual & Biofeedback Training automatically ends with the conclusion "Training Complete"



Hypoxic Test

Abortion by Operator

Relaxation Phase (3 min)

ONLY Interrupted Manual & Biofeedback Trainings





Conclusions

Manual/Biofeedback

Hypoxic Test


Interrupted Sessions

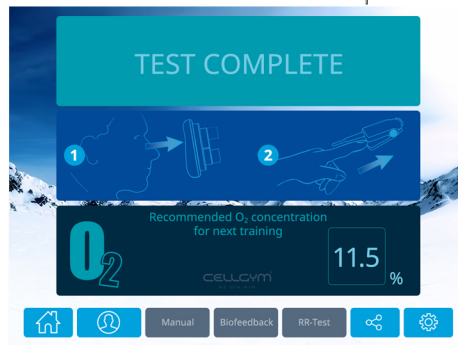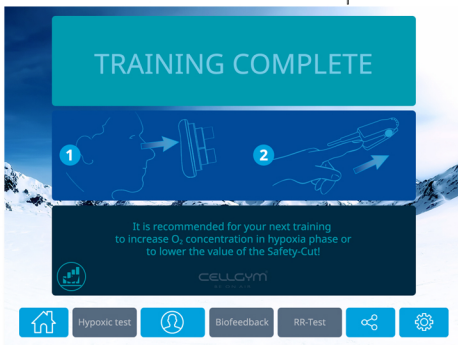

Feedback / Recommendation for next Training


Recommendation for O2 Concentration for next training

Bottom Menu on mask Conclusion

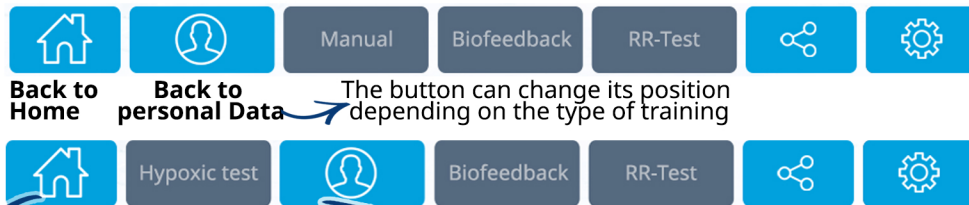

Back to Home **Back to personal Data** — The button can change its position depending on the type of training

Back to „Home“ for next patient

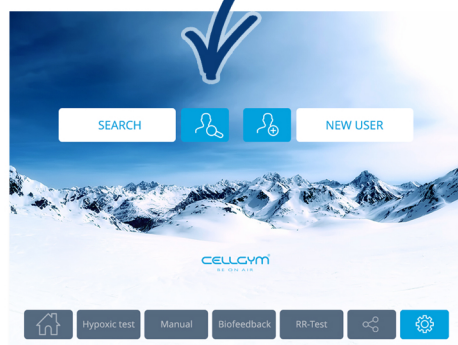

Back to personal data of the current user.

Notes for current/next training can be saved.



Back to select existing user or create new user

Back to the current patient data screen